KULLANIM TALİMATI

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

INSTRUCTIONS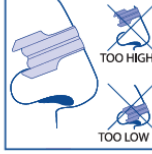

This is how the strip should be placed for best results.


1 Wash and dry your nose to remove dirt and oils

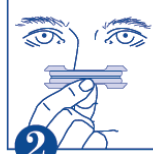2 Remove protective liner and apply as shown.

3 Gently rub to secure.

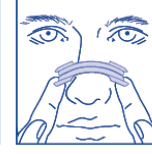Always remove the strip **SLOWLY**. Do this while washing your face or showering with warm water. Loosen the ends, then gently lift both sides.

Breathe Right by OtriNatura Nasal Strips Opens your nose to,

- Breathe better,
- Relieve nasal congestion,
- Reduce snoring

GSKTR-BR-KK001

How they work.

- Two flexible pieces gently lift upwards.
- Special adhesive comfortably secures strip.
- Nasal passages are "lifted" open.

Helpful Hints

- Do not touch adhesive.
- Use an astringent to clean oily skin.
- If skin is dry after use, try a skin moisturiser.
- For dry or sensitive skin, use Breathe Right Clear strips.
- For sensitive skin try a drop of lotion under the center of the strip (only the ends need to adhere for the strip to work)
- For exercise/sports: Apply 30 minutes before you begin to improve adhesion when you perspire.

CAUTIONS: For external use only. Do not use over sores, sunburn or irritated skin. Do not use for more than 12 hours per day. Disposable - use each strip only once. If rash or skin irritation occurs, discontinue use. Not for children under 5 years of age. Packaging of this product contains natural rubber latex which may cause allergic reactions. Can be used day or night. Keep out of reach of children. Store at room temperature.

Sleep apnea: Breathe Right by OtriNatura nasal strips are not intended to treat Sleep Apnea. Persons who have symptoms of Sleep Apnea should consult their doctor.

Adverse reactions to Breathe Right by OtriNatura nasal strips are mainly local application site reactions. Proper removal of strips will decrease the likelihood of application site reactions. Examples of voluntarily reported cases application site reactions (e.g. erythema, irritation, pain, exfoliation, excoriation, bleeding, scab, and discolouration) and hypersensitivity reactions including rash, pruritus and facial edema.

FOR SINGLE USE ONLY

GSK Consumer Healthcare (GMDT) Clocherane, Youghal Road, Dungarvan, Co. Waterford, İrlanda

Breathe Right by OtriNatura Nasal Strips is a registered trademark of the GlaxoSmithKline group of companies.

TR 943930 - 31046117